




AryLift

Harness Connection Quick Reference Guide

For more information, please visit www.AryLift.101Mobility.com

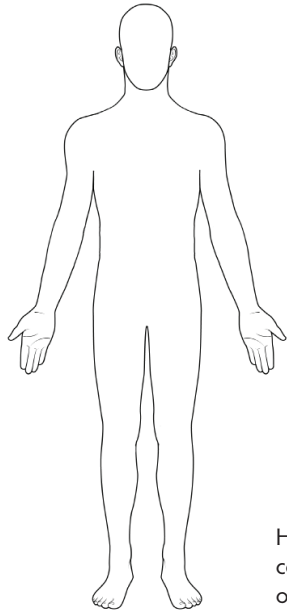

← Strap 1

← Strap 2

← Strap 3

Harness straps are always connected in the direction of head to foot.


Find the application desired and the corresponding strap color. Hook the applicably colored strap onto the lifting bars.

Important: Identification tags will be found on all harnesses. Always put this tag underneath the pillow and downward facing the bed.

NOTE: Always use caution and pay close attention!!! Make sure all hooks and straps are hooked securely, make sure the bars or lifting straps do not get caught on anything, and make sure your patient is comfortably positioned.



	Shoulders	Waist	Legs	Notes
Position 1: Reclining Position (Reverse Trendelenburg position)	Black	Blue	Blue	
Position 2: Trendelenburg Position (Feet above head)	Long White	Standard White	Standard White	Lift patient just a few inches above bed
Position 3: Level Position	Black	White	White	
Position 4: Wheel Chair Transfer (Sitting)	Black	Black	-	Lift patient just a few inches above bed; Slide the shoulder hook and waist hook together enough to connect the WHITE shoulder strap to the waist hook; Continue to raise the patient to the desired height; Then roll the machine over the wheel chair and lower.

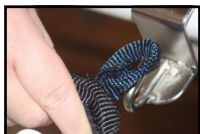

Blue Strap



Black Strap



White Strap



Long White Strap



Position 1



Position 2



Position 3



Position 4